



PRODUCT GUIDE

Glue-U Adhesives is a Dutch company making innovating products for farriers and horses since 2006. Following our philosophy *“together we take care of your horse”* we designed special adhesives to glue any kind of horseshoes, acid free silicones and very fast urethane fillers for any purpose. All Glue-U products are Dutch engineered and manufactured both in Europe and USA: same quality, same performance.

Glue-U Adhesives BV

Droogdokkeneiland 8
5026SR Tilburg
The Netherlands

Tel. +31 (0)13 54 53 118

E-mail: sales@glue-u.com

Website: www.glue-u.com

TABLE OF CONTENTS

6

SHUFILL CUSHION

SUPER SOFT A20	8
SOFT A30	10
MEDIUM A50	12
FIRM A60	14

16

SHUFILL SILICONES

SOFT A10	18
MEDIUM-SOFT A20	20
MEDIUM-SOFT TRANSPARENT A20	22
MEDIUM A30	24
FIRM A40	26

28

SHUFILL HOOV PACKING

SUPER SOFT A15	30
SOFT A25	32
MEDIUM SOFT A35	34

36

SHUFIX

BLACK / WHITE	38
---------------	----

40

SHUFIT

BLACK / BEIGE	42
---------------	----

44

PRE FIX

48

CARTRIDGES

STORAGE	48
USE	49

50

SHUPADS

52

ACCESSORIES

56

GLUSHU

MINI GLUSHU	58
APPLICATIONS	59

SHUFILL CUSHION

Firm but soft urethane pad material in 4 durometers
A20 Super Soft – A30 Soft – A50 Medium – A60
Firm. 250cc cartridges fitting in standard caulking
guns. Offers protection, cushioning and support for
the hoof sole. Fill up to ground level for support of
foundered and laminitis cases. Usage of supplied
meshes is recommended, but not obliged. *Set time
approx. 35 seconds. Temperature and humidity might
have effect on the performance of this product.*



SUPER SOFT A20

Product Number	200120250
Description	<i>Super Soft pad material with adhesion</i>
Cartridge	250cc
Color	<i>Cristal Clear</i>
Set time	<i>35 seconds</i>

SOFT A30


Product Number	200130250
Description	<i>Soft shock absorbing pad material with adhesion</i>
Cartridge	250cc
Color	<i>Cristal Clear</i>
Set time	<i>35 seconds</i>

MEDIUM A50

Product Number	200150250
Description	<i>Soft & supportive pad material with adhesion</i>
Cartridge	250cc
Color	<i>Cristal Clear</i>
Set time	<i>35 seconds</i>


FIRM A60

Product Number	200160250
Description	<i>Supportive pad material with adhesion</i>
Cartridge	250cc
Color	<i>Cristal Clear</i>
Set time	<i>35 seconds</i>

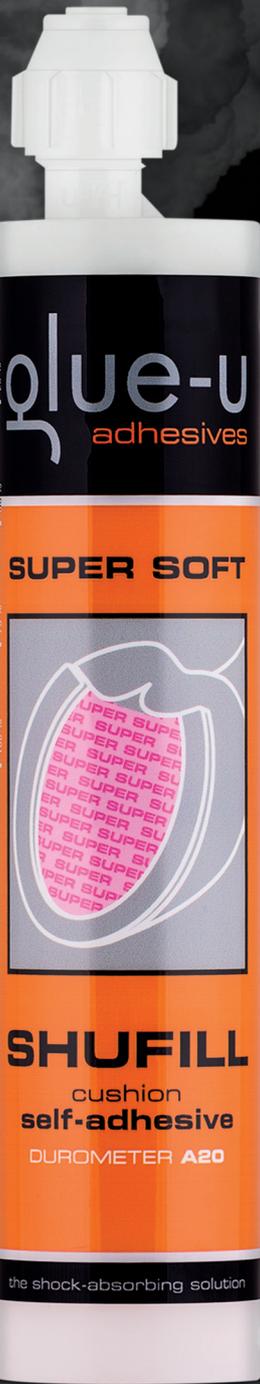

glue-u
adhesives

SUPER SOFT

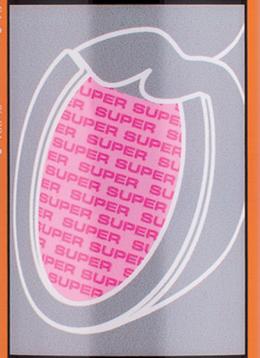

SHUFILL
cushion
self-adhesive
DUROMETER A20

the shock-absorbing solution

the shock-absorbing solution
cushion
self-adhesive
SHUFILL

SHUFILL CUSHION

SUPER SOFT A20

Is a 35 second fast setting urethane product to be used as hoofpad material, offering the farrier the advantage to prevent dirt clogging between sole and pad. Since it is super soft we advice to use it in combination with Shupad.

FEATURES:

- Very soft filling material
- Stays soft even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time becomes hot during setting (in case of sensitive feet use Shufill silicones)

STANDARD APPLICATION:


- With pads

SUPERSOFT WITH PAD



SEE POSSIBLE
APPLICATIONS



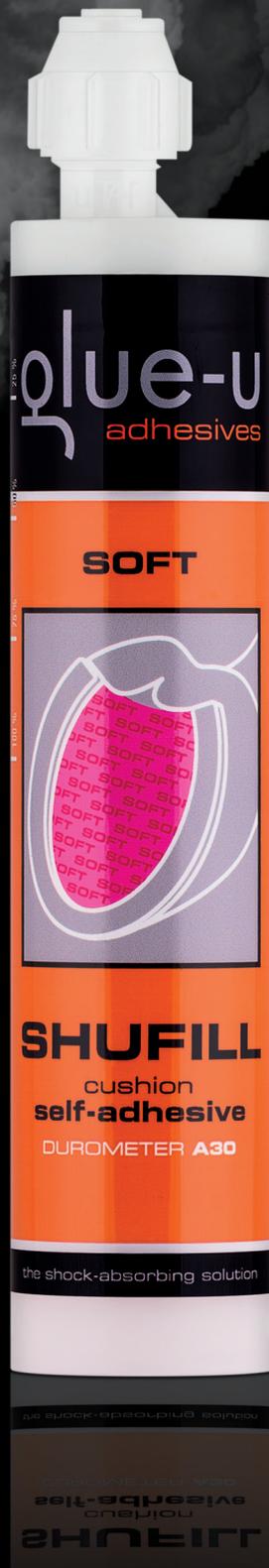

SHUFILL CUSHION

SOFT A30

Is a soft fast setting hoofpad material offering more comfort for the horse. It can be used with or without pads for filling as well as shock absorbing purposes. It can also give great benefits for foundered horses.

FEATURES:

- Soft filling / shock absorbing material
- Stays soft even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time becomes hot during setting (in case of sensitive feet use Shufill Silicones)

STANDARD APPLICATION:

- With pads

ALTERNATIVE APPLICATION:


- With mesh filled to ground level

SOFT WITH MESH



WATCH ALL
CUSHION FEATURES



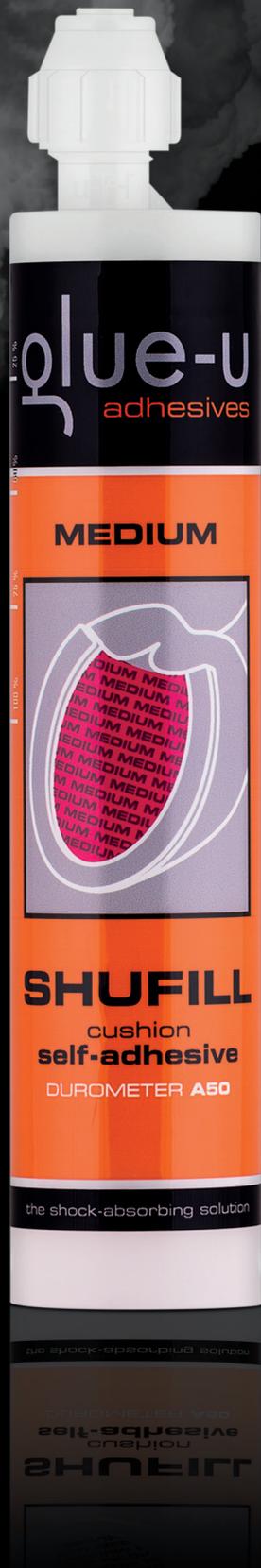

glue-u
adhesives

MEDIUM



SHUFILL

cushion
self-adhesive

DUROMETER A50

the shock-absorbing solution

the shock-absorbing solution

cushion
self-adhesive

SHUFILL

DUROMETER A50

SHUFILL CUSHION

MEDIUM A50

Is a shock absorbing pad material with a setting time of 35 seconds often used for sport horses. It can be used with pad or mesh. However it is advisable to use a mesh for the best result.

FEATURES:

- Medium hard shock absorbing material
- Stays stable during shoeing period, even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time
- Becomes hot during setting (in case of sensitive feet use Shufill Silicones)

STANDARD APPLICATION:

- With pads

ALTERNATIVE APPLICATION:

- With mesh

MEDIUM WITH MESH AND FOAM PAD




CHECK OUT THIS
APPLICATION VIDEO





glue-u
adhesives

FIRM

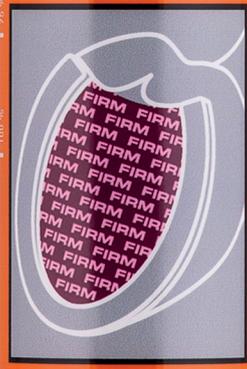

SHUFILL
cushion
self-adhesive
DUROMETER A60

the shock-absorbing solution

the shock-absorbing solution
self-adhesive
cushion
SHUFILL

SHUFILL CUSHION

FIRM A60

Shufill Cushion Firm is a supporting pad material which can be used on horses with shoes or barefoot. Thanks to the support on the sole it relieves the pressure on the wall and improves heel growth.

FEATURES:

- Hard supporting pad material
- Stays stable during shoeing interval, even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time
- Becomes hot during setting

STANDARD APPLICATION:

- With pads

ALTERNATIVE APPLICATION:

- With mesh

FIRM AS PROTECTION LAYER



SHUFILL SILICONES

Firm but soft silicone pad material in 5 versions A10 (blue) – A20 (green) – A20 (transparent) – A30 (purple) – A40 (yellow). 250cc cartridges fitting in standard caulking guns. Offers protection, shock absorption and support for the hoof sole. Always use Shufill Silicones in combination with Shupads! Shufill Silicones do not contain acids or chemicals and are therefore very suitable for wound treatment. *Set time 4 minutes, depending on weather conditions. Heating up Shufill Silicones before use speeds up setting time significantly. Temperature and humidity might have effect on the performance of this product.*



SOFT**A10**

Product Number	210105250
Description	<i>Extremely Soft Silicone pad material</i>
Cartridge	250cc
Color	<i>Blue</i>
Set time	<i>4 minutes (heated 2 minutes)</i>

MEDIUM-SOFT**A20**

Product Number	210107250
Description	<i>Very Soft Silicone pad material</i>
Cartridge	250cc
Color	<i>Green</i>
Set time	<i>4 minutes (heated 2 minutes)</i>

MEDIUM-SOFT**A20**

Product Number	210103250
Description	<i>Very Soft Silicone pad material</i>
Cartridge	250cc
Color	<i>Transparent</i>
Set time	<i>4 minutes (heated 2 minutes)</i>

MEDIUM**A30**

Product Number	210109250
Description	<i>Soft Silicone shock absorbing pad material</i>
Cartridge	250cc
Color	<i>Purple</i>
Set time	<i>4 minutes (heated 2 minutes)</i>

FIRM**A40**

Product Number	210104250
Description	<i>Medium hard Silicone supportive pad material</i>
Cartridge	250cc
Color	<i>Yellow</i>
Set time	<i>4 minutes (heated 2 minutes)</i>



SHUFILL SILICONES

SOFT A10

Is a silicone product to be used as hoofpad material, offering the farrier the advantage to prevent dirt clogging between sole and pad. Can only be used as filling.

FEATURES:

- Very soft filling material
- Stays soft even in cold weather conditions
- No adhesion to the sole
- 4 minutes setting time
- If heated upto body temperature setting time becomes 2 minutes
- Keeps low temperature during setting
- Acid free

STANDARD APPLICATION:

Always with pads

DIFFERENT DENSITIES


A10 blue for filling +

A20 green for light shock absorption



CHECK OUT THIS
APPLICATION VIDEO



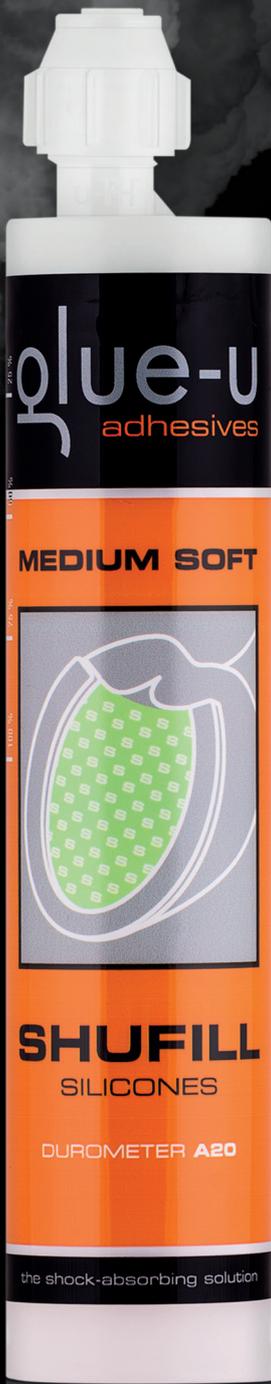

glue-u
adhesives

MEDIUM SOFT



SHUFILL
SILICONES

DUROMETER A20

the shock-absorbing solution

the shock-absorbing solution

SILICONES

SHUFILL

SHUFILL SILICONES

MEDIUM-SOFT A20

Is a silicone product to be used as hoofpad material, offering the farrier the advantage to prevent dirt clogging between sole and pad. It can be used as filling or light shock absorption.

FEATURES:

- Soft filling / light shock absorbing material
- No adhesion
- Stays soft even in cold weather conditions
- 4 minutes setting time (2 minutes when heated)
- Keeps low temperature during setting
- Acid free

STANDARD APPLICATION:


- Always with pads

FULL FILLING WITH SHUPAD FROG SUPPORT



SEE POSSIBLE
APPLICATIONS



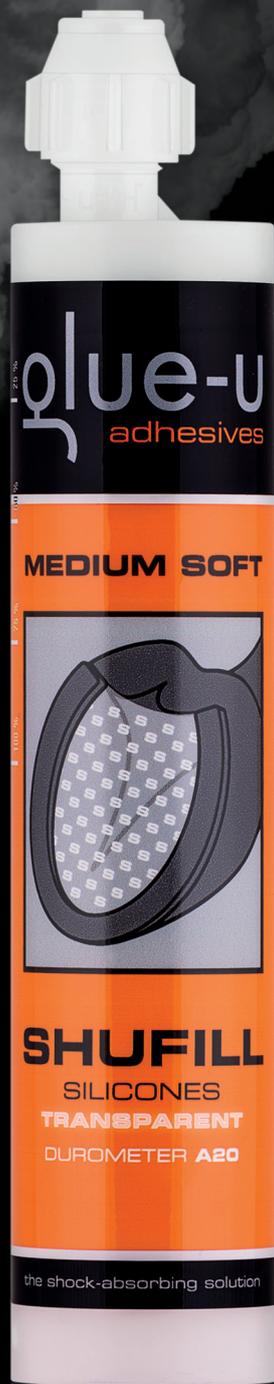

SHUFILL SILICONES

MEDIUM-SOFT A20 TRANSPARENT

Is a transparent silicone product to be used as hoofpad material, offering the farrier the advantage to prevent dirt clogging between sole and pad. It can be used as filling or light shock absorption.

FEATURES:

- Soft filling / light shock absorbing material
- No adhesion
- Stays soft even in cold weather conditions
- 4 minutes setting time (2 minutes when heated)
- Keeps low temperature during setting
- Acid free

STANDARD APPLICATION:


- Always with pads

FULL FILLING WITH SHUPAD




WATCH ALL
SILICONE FEATURES



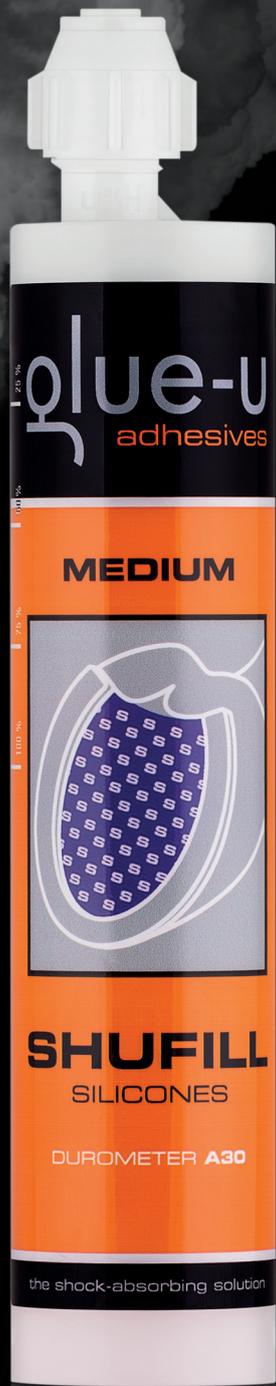

glue-u
adhesives

MEDIUM

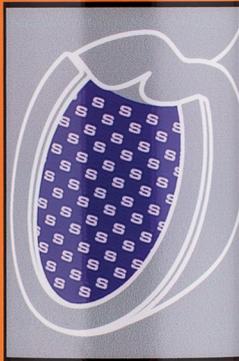

SHUFILL
SILICONES

DUROMETER A30

the shock-absorbing solution

the shock-absorbing solution

SILICONES
SHUFILL

SHUFILL SILICONES

MEDIUM A30

This silicone hoofpad material is mostly used for shock absorption. It offers more comfort to sport horses and also reduces stress on the feet for foundered horses.

FEATURES:


- Medium shock absorption material
- No adhesion
- Stays soft even in cold weather conditions
- 4 minutes setting time (2 minutes when heated)
- Keeps low temperature during setting
- Acid free

STANDARD APPLICATION:

- Always with pads

FULL FILLING WITH SHUPAD



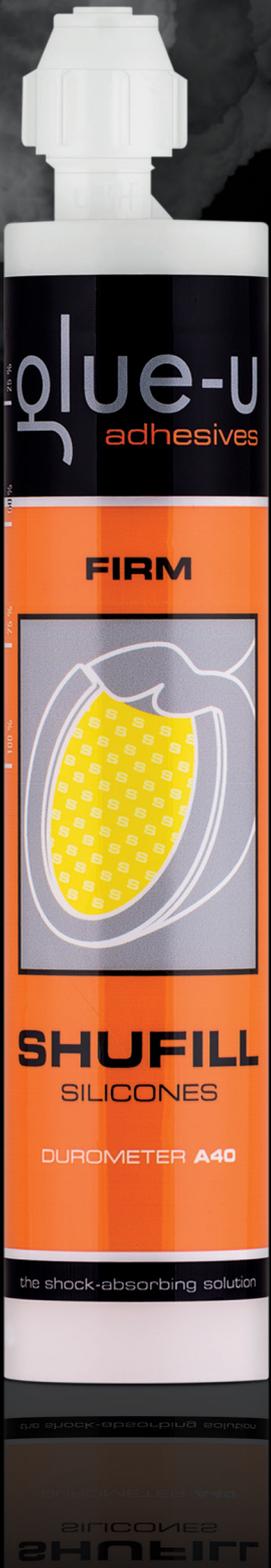

glue-u
adhesives

FIRM



SHUFILL
SILICONES

DUROMETER A40

the shock-absorbing solution

the shock-absorbing solution
SILICONES
SHUFILL

SHUFILL SILICONES

FIRM A40

The firm hoofpad material offers the best support for sport horses. The shock absorbing feature gives the best comfort for jumping horses and horses riding on hard surfaces.

FEATURES:

- Medium hard light supportive material
- No adhesion
- Stays soft even in cold weather conditions
- 4 minutes setting time (2 minutes when heated)
- Keeps low temperature during setting
- Acid free

STANDARD APPLICATION:

- Always with pads

OPTIMAL SUPPORT WITH SHUPAD



SHUFILL

HOOF PACKING

Lightweight hoof filler for shock absorption and protection. Mixed by hand. Available in 2×660 gr / 1.5 lb or 2×2.5 kg / 5.5 lb and durometers A15 (blue) – A25 (pink) – A35 (green). Specially for protecting inner hoof and solar surface. Easily mixable and shapeable by hand. Shufill Hoof packing is a non irritating product. Easy to cut which makes it applicable in many situations. Shufill Hoof Packing does not contain acids or chemicals and is therefor very suitable for wound treatment. *Temperature and humidity might have effect on the performance of this product. Heating up Shufill hoof packing before use speeds up setting time significantly.*



EXTRA SOFT A15

Product Number	<i>250105660 (660 gr) / 250105025 (2500 gr)</i>
Description	<i>Extra Soft Silicone pad material mixed by hand</i>
Jar / Bucket	<i>2x660 gr/1.5 lb / 2x2500 gr/5.5 lb</i>
Color	<i>Blue</i>
Set time	<i>4-5 minutes (heated 2-3 minutes)</i>

SOFT A25

Product Number	<i>250106660 (660 gr) / 250106025 (2500 gr)</i>
Description	<i>Soft Silicone pad material mixed by hand</i>
Jar / Bucket	<i>2x660 gr/1.5 lb / 2x2500 gr/5.5 lb</i>
Color	<i>Pink</i>
Set time	<i>4-5 minutes (heated 2-3 minutes)</i>

MEDIUM A35

Product Number	<i>250107660 (660 gr) / 250107025 (2500 gr)</i>
Description	<i>Medium Silicone pad material mixed by hand</i>
Jar / Bucket	<i>2x660 gr/1.5 lb / 2x2500 gr/5.5 lb</i>
Color	<i>Green</i>
Set time	<i>4-5 minutes (heated 2-3 minutes)</i>



SHUFILL HOOF PACKING

A15 BLUE

Is an extra soft impression material to be mixed by hand. It's mostly used for filling, light shock absorption or just to give more comfort to the horse. Since the product is very soft it is only possible to fill complete soles. In case of laminitis or other cases where there is an extremely painful area of the sole that needs to be isolated, the impression material can be trimmed away once it is cured after applying the shoe and pad.

FEATURES:

- Very soft impression material
- Non toxic
- Skin friendly (not irritating or causing allergic reactions)
- Setting time 4-5 min (heated 2-3 min)
- Easily mixable and shapeable by hand
- Can be combined with any medical application to the sole

STANDARD APPLICATION:

- Always with pads

ALTERNATIVE APPLICATION:

- Medical plate

MEDICAL PLATE



CHECK OUT THIS
APPLICATION VIDEO





SHUFILL HOOF PACKING

A25 PINK

Is a soft impression material and it is the most used version of this product range. It gives a perfect shock absorption to the complete sole or locally as frog support in combination with for example a heartbar shoe.

FEATURES:

- Soft impression material
- Non toxic
- Skin friendly (not irritating or causing allergic reactions)
- Setting time 4-5 min (heated 2-3 min)
- Easily mixable and shapeable by hand
- Can be combined with any medical application to the sole

STANDARD APPLICATION:

- Full filling with pads

ALTERNATIVE APPLICATION:

- Locally on bar shoes (Heartbar, Straightbar)

FULL FILLING WITH PADS



SEE POSSIBLE
APPLICATIONS





SHUFILL HOOF PACKING

A35 GREEN

Is your best partner to support the hoof completely or just locally at the frog. Thanks to the rigid end product it's very easy to use with all kinds of barshoes or pads.

FEATURES:

- Rigid impression material
- Non toxic
- Skin friendly (not irritating or causing allergic reactions)
- Setting time 4-5 min (heated 2-3 min)
- Easily mixable and shapeable by hand
- Can be combined with any medical application to the sole

STANDARD APPLICATION:

- Full filling with pads

ALTERNATIVE APPLICATION:

- Locally on bar shoes (Hearthbar, Straightbar)

FULL FILLING WITH PADS



WATCH ALL
HOOFPACKING FEATURES



SHUFIX

FAST SET

Urethane adhesive that is very suitable for making extensions on foals' hoofs and custom plastic shoes. Also suitable for retouching or rebuilding heels, making medial/lateral corrections and for smaller hoof repairs. The hoof should be clean and especially dry. 250cc cartridges fitting in standard caulking guns. *Curing time between 30 – 60 seconds, workable after about 5 minutes. Temperature and humidity can influence the performance of this product.*



BLACK

Product Number	150102250
Description	<i>Instant crack repair super fast setting / on foal extensions</i>
Cartridge	250cc
Color	Black
Set time	30-60 seconds

WHITE

Product Number	150101250
Description	<i>Instant crack repair super fast setting / on foal extensions</i>
Cartridge	250cc
Color	White
Set time	30-60 seconds



SHUFIX BLACK / WHITE

Shufix is an ultra fast horse glue which is very suitable for making extensions on foals' hoofs and custom plastic shoes. Also suitable for retouching or rebuilding heels, making medial/lateral corrections and for smaller hoof repairs. In severe winter circumstances Shufix can also be used to glue most kind of glue-on shoes. Since it's a urethane product, the hoof should be dry and clean before applying Shufix.

FEATURES:

- Available in black and white
- Super fast setting time (30 - 60 seconds)
- Easy to rasp
- Hard but flexible material
- High temperature while setting (keep +/- 1 cm / 0.4") from coronary band)

STANDARD APPLICATION:

- Hoof extensions

ALTERNATIVE APPLICATION:

- Glue-on shoes
- Crack repairs in severe winter circumstances
- Temporary protection
- Heel lifts

HOOF EXTENSION



CRACK REPAIR
APPLICATION VIDEO



SHUFIT

GLUE ON SHOE

Acrylic glue for bonding horseshoes and hoofwall repairs. Available in 150cc and 420cc black and beige. Fits in standard caulking gun using an adapter or just use the Glue-U dispenser for 150 cc cartridges. 420cc only usable with special gun. Bonds steel, aluminium and even urethane shoes. Suitable for repairing large cracks and creating extensions. *Initial set time: 90-120 seconds (open time). Weight-bearing in 4-5 minutes (even in cold conditions). Temperature and humidity might have effect on the performance of this product. Heating up Shufit before use speeds up setting time significantly.*



BLACK

Product Number	100102150 / 100102420
Description	Glue-on-shoe / Hoof repair
Cartridge	150cc / 420cc
Color	Black
Set time	4-6 minutes / weight bearing 7-8 minutes Heated; 120-180 seconds

BEIGE

Product Number	100114150 / 100114420
Description	Glue-on-shoe / Hoof repair
Cartridge	150cc / 420cc
Color	Beige
Set time	90-120 seconds / weight-bearing 4-5 minutes Heated; 45-75 seconds



SHUFIT BLACK / BEIGE

Shufit acrylic glue is the most versatile glue used in the horse industry. It can be used to glue-on every kind of shoe or do repairs from the smallest crack to the most extreme hoof wall problems. Thanks to its great features it's mixable by hand or easy to use with Glue-U mixing tips for the perfect mixture. Shufit can even be moved from one place to another while setting. For example moving excess material to a spot where extra glue is needed.

FEATURES:

- Available in black and beige
- 150cc cartridges usable in standard caulking guns
- Mixable by hand or with mixing tips
- Very high adhesion
- Easy to rasp
- Hard but flexible material
- Setting time can be speed up when heated to body temperature

STANDARD APPLICATION:

- Glue-on shoes

ALTERNATIVE APPLICATION:

- Crack repairs
- Hoof repairs

GLUE-ON GLUSHU



CHECK OUT THIS
APPLICATION VIDEO



PRE FIX SUPER GLUE

Pre fix is a system to bond a GluShu to the foot just to keep it in place. It is a 2 component super glue which will bond the GluShu to the hoof in seconds. This allows you to do the actual glueing of the shoe with the foot on the ground. Pre fix is very useful if you have a very nervous horse or a horse that is unable to stand on three legs (laminitis or injury). *Pre fix is no structural glue. It is temporarily only to keep the GluShu in place for the time you need until the actual glue is cured. One component is the activator spray, the other component is a transparent gel in a squeeze bottle.*



ACTIVATOR

Product Number	490100200
Description	<i>Pre glue Glushu in combination with Pre fix CA glue</i>
Cartridge	<i>200cc / 6.76 oz</i>
Color	<i>Transparent</i>
Set time	<i>15 sec</i>

CA GLUE

Product Number	490101100
Description	<i>Pre glue Glushu in combination with Pre fix Activator</i>
Cartridge	<i>150 gr / 5.29 oz</i>
Color	<i>Tranparent</i>
Set time	<i>15 sec</i>



PRE FIX

Pre fix is used in 2 steps. Once you fitted and shaped the GluShu and you are ready to glue, apply the Pre fix glue according to our instruction video and put down the foot for the actual glueing with Shufit once the Glushu is bonded to the hoof.

FEATURES:

- 15 seconds fast setting superglue
- Bonds once 2 components are in contact

STANDARD APPLICATION:

- Only for pre-application of Glushu shoes

PRE FIX GLUSHU



CHECK OUT THIS
APPLICATION VIDEO



CARTRIDGE STORAGE

DO'S

Protect from freezing.

Store Shufit at approx. 10 °C / 40 °F.

Store Poly-urethanes and silicones at a temperature of approx. 15°C / 60°F.

Store the products at a dry place.

"First in First out" use.

Keep cartridges upright.

Possibility to use mixing tip as cap.

DONT'S

Store the product in constant changing temperatures.

Put used cartridges back at the stock.

Open the cartridge for a couple of minutes.

Leave cartridge open after use.

CARTRIDGE USE

DO'S

Check the level of both components.

Make sure both canals are free.

Before actual use, first press just a little bit in a cup to ensure proper mixing before installing the mixing tip.

Make sure the product is mixed properly before applying on the foot.

Work in open well ventilated space.

Wear protective clothing + gloves.

Trim the hoof as normal. Trimming is the most important part of your job.

While glueing, keep the foot up until glue is completely cured. (setting time depends on temperature and humidity)

DONT'S

Do not smell or breath-in the product directly.

Contact with skin. Product might cause allergic reactions.

Don't use on wounds or open skin.


Use too much glue at once. More glue is more heat.

Avoid glue on the sole and frog.

Do not heat up cartridges in hot water. Air heating only.

SHUPADS

The base for silicones and impression materials. These transparent pads are available in 4 different types. M and XL flat, L flat and wedge with frog support. It's easy to cut the pads and fit to every hoof. Use our size chart to determine the size closest to what you need. The high quality urethane material ensures no limitation for the hoof mechanism and keeps pad materials in position. *Experiences proved that Glue-U Shupads are extra durable and can be used for a long period. Every Shupad is designed with self closing injection holes. So filling up the hoofs becomes easier with Glue-U Shupads.*

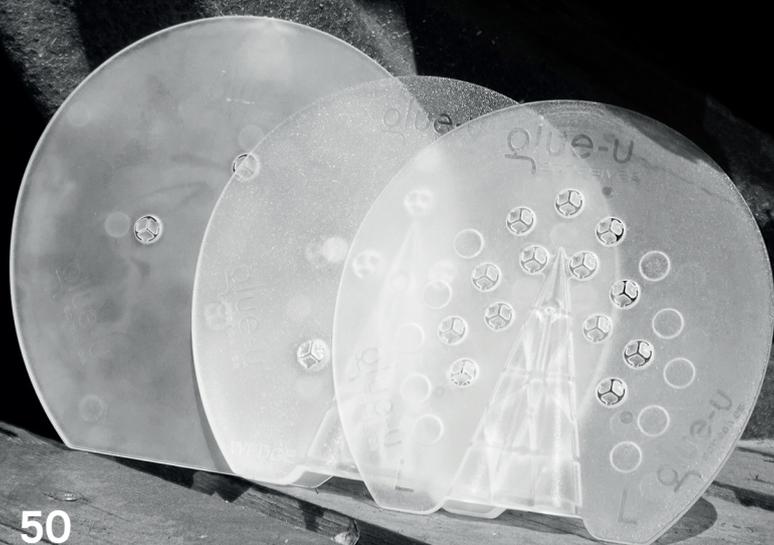

FLAT FLEX

Product Number	810050010
Description	Flat pad with 4 filling holes
Volume	10 pcs (Medium)
Color	Transparent
Dimensions	150 x 155 mm 5.9 x 6.1"

FROG SUPPORT FLEX

Product Number	810060010
Description	Flat pad with multiple filling holes and frog support
Volume	10 pcs (Large)
Color	Transparent
Dimensions	165 x 165 mm 6.5 x 6.5"

FLAT FLEX

Product Number	810070010
Description	Flat pad with 4 filling holes
Volume	10 pcs (Extra Large)
Color	Transparent
Dimensions	180 x 180 mm 7.08 x 7.08"

WEDGE

Product Number	810060006
Description	Wedge pad with multiple filling holes and frog support
Volume	6 pcs (Large)
Color	Transparent
Dimensions	165 x 165 mm 6.5 x 6.5" 3° wedge; 10 mm 0.4"



ACCESSORIES

TOOLS



ADAPTER

Product Number	490002001
Description	To fit 150cc Shufit in a normal dispenser
Volume	1 pc



DISPENSER FOR SHUFIT 150cc

Product Number	827100150
Description	Dispenser
Volume	1 pc



DISPENSER FOR SHUFIT 420cc

Product Number	827100420
Description	420cc dispenser
Volume	1 pc



DISPENSER UNIVERSAL

Product Number	820000000
Description	150 & 250cc dispenser
Volume	1 pc



DRAW KNIFE

Product Number	490000002
Description	Knife to remove GluShu
Volume	1 pc



SELF ADHESIVE FOAM BOARD

Product Number	810001020
Description	Foam board with adhesion 199 x 199 mm / 7.85 x 7.85"
Volume	20 pcs



HAND WRAPPING FOIL

Product Number	890503001
Description	Hoofwrap foil
Volume	1 pc



HAND WRAPPING FOIL HOLDER

Product Number	890512001
Description	Hoofwrap foil holder
Volume	1 pc



HEATER 150CC & 250cc / 420cc

Product Number	890312150 / 890312420
Description	Glue heating box 12V, 110V & 220V
Volume	1 pc



MESH

Product Number	892002010
Description	Mesh for Shufill Cushion 2500 x 200 mm / 98.5 x 7.9"
Volume	1 pc




MIXING TIP

Product Number	610007010
Description	Green square mixing tip
Volume	10 pcs



SHUCLEANER

Product Number	490000750
Description	Aceton free hoof cleaner
Volume	750cc

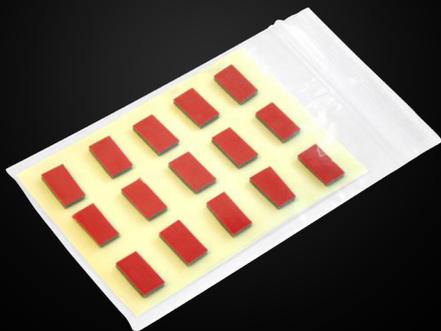

SPACER TAPE

Product Number	891002010
Description	Special tape to keep space between the hoof and horseshoe
Volume	15 pcs



SHUFIT DYE

Product Number	891002001
Description	Black pigment to dye Shufit beige to black
Volume	1 pc



CORRECTION PLATES

Product Number	420290075 / 420290085 / 420290095
Description	Mini Glushu extension plates for all kinds of extensions
Volume	2 pcs



INSERTS FOR GLUSHU & MINI GLUSHU

Product Number	Available in every size
Description	Frog support, wedges, closed pads to fit Glushu
Volume	2 pcs

GLUSHU

GLUE-ON HORSESHOE

Glushu is a flexible cuff bonded to a specially designed aluminium horse shoe. This gives the horse a light, hoof supporting, cushioned shoe which eliminates any stress or damage to the horse's hoof and is easy to apply. GLUSHUS are non invasive, completely eliminating the risk of hoof injury due to nailing. Easy and fast to apply. Available in pairs or kit. *Uses a fast acting glue. We strongly advise you to use Glue Shufit glue which is specifically designed to be used with GluShu. The cuff allows the shoe to be glued in stages. No need for any special equipment. Simple sizing system & large range of sizes. No alteration needed. Eliminates the risk of stress when nailing into a thin white line or poor hoof wall. Cushions the feet. The cuff can be used to hold topical medication in place. Prevents further damage to hoof wall.*



GLUSHU KITS & PAIRS

Kits including all the essential equipment needed to glue on a pair of Glushus;

- 1x GluShu Pair
- 1x Shufit 150cc
- 1x Adapter
- 4x Mixingtip

KIT SIZE	Product number
4x0 Front	410290040
3x0 Front	410290030
00 Front	410290020
0 Front	410290010
1 Front	410290001
2 Front	410290002
3 Front	410290003
4 Front	410290004
2x0 Hind	410291020
0 Hind	410291010
1 Hind	410291001
2 Hind	410291002
3 Hind	410291003

PAIR SIZE	Product number
4x0 Front	400290040
3x0 Front	400290030
00 Front	400290020
0 Front	400290010
1 Front	400290001
2 Front	400290002
3 Front	400290003
4 Front	400290004
2x0 Hind	400291020
0 Hind	400291010
1 Hind	400291001
2 Hind	400291002
3 Hind	400291003

MINI GLUSHU FOAL SHOE

Mini Glushu is a smaller version of the standard Glushu applicable for as well front as hind feet. This glue-on shoe is a very easy to use glue-on shoe for foals and Shetland ponies. It is a versatile shoe with many options. It can easily be combined with the Mini Glushu extension plates or inserts.

For users only applying Mini Glushus we also offer an application kit including all you need to glue on about 5 sets of Mini Glushu.

Application kit;
1x Shufit 150cc
1x Adapter
1x Pre fix set
10x Mixingtip

PAIR SIZE	Product number
75 mm / 2.95"	400290075
85 mm / 3.35"	400290085
95 mm / 3.74"	400290095

PRODUCT	Product number
Application kit	410290000
Correction plates	420290075 420290085 420290095

APPLICATIONS

Glushu is a glue-on shoe with a very similar use compared to normal steel/aluminium shoes. It can be applied in every situation where nailing is difficult, impossible or not wanted. Glushu is glued to the hoof wall. Which means that limitation of hoof mechanism is minor. Glushu can also be combined with all Glue-U Shufill Cushions or Glushu inserts combined with Shufill Hoof packing or Silicones. Glushu can easily be reshaped, grinded, etc...


STANDARD APPLICATION:

Glushu front or hind complete shoes. In case needed combined with Shufill Cushion or inserts.

SEE LINK FOR APPLICATION VIDEO - PAGE 43



MODIFICATION
INSTRUCTION VIDEO

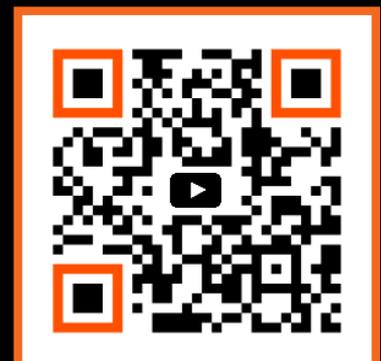



ALTERNATIVE APPLICATION:

Glushu front shoe + front cuff cut out to reposition breakpoint for better roll-over. In case needed Glushu ground surface can also be grinded to a round shape.

ALTERNATIVE APPLICATION:

Glushu Hind shoe + front cuff cut out to use as open toe shoe. Heels can also be grinded to round shape for better roll-over.



ALTERNATIVE APPLICATION:

Glushu combined with mesh or pad for filling with pad materials. Mesh or pad glued right before application.

*MINI GLUSHU
EXTENSION PLATES:*

Mini Glushu adjusted with extension plates prior to glueing on for any kind of hoof stand corrections.




*MINI GLUSHU
APPLICATION:*

Mini Glushu combined with a pad and Shufill Hoofpacking to reduce as much as possible stress on the sole. The damaged toe is kept open to assure daily disinfection and cleaning.

*GLITTER GLUSHU
APPLICATION:*



That moment when you get the request to make it a glitter Glushu. This is just a Glushu special for those who like to answer to the glitter request. Anika Ridgeway shows how to modify Glue-U Shufit to the desired look.

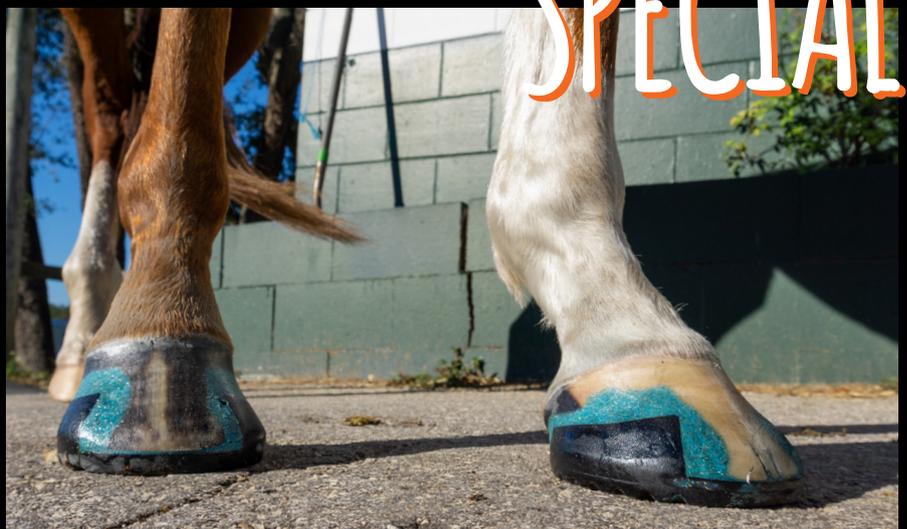

Glue-U Adhesives BV

Droogdokkeneiland 8
5026SR Tilburg
The Netherlands

Tel. +31 (0)13 54 53 118

E-mail: sales@glue-u.com

Website: www.glue-u.com