

SHUPACK

SOFT A30

- Soft filling / shock absorbing material
- Stays soft even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time becomes hot during setting (in case of sensitive feet use SHUSIL)



SHUPACK soft A30 is a soft fast setting hoofpad material offering more comfort for the horse. It can be used with or without pads for support as well as shock absorbing purposes. It can also give great benefits for foundered horses.

Comes in 250cc cartridges fitting in standard caulking guns. Fill up to ground level for support of foundered and laminitis cases. When used without pads, usage of meshes is recommended, but not obliged. Set time approx. 35 seconds. Temperature and humidity might have effect on the performance of this product.

PRODUCT APPLICATION

Standard application

- With pads

Alternative application

- With mesh filled to ground level

UPC



TECHNICAL PRODUCT DATA

Product Number	200130250
Description	Soft shock absorbing pad material with adhesion
Cartridge	250 cc
Color	Cristal Clear
Set time	35 seconds

Glue-U Adhesives BV

Droogdokkenland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN

V.003.US

SHUPACK

MEDIUM A50

- Medium hard shock absorbing material
- Stays stable during shoeing period, even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time
- Becomes hot during setting (in case of sensitive feet use SHUSIL)



SHUPACK A50 is a shock absorbing pad material with a setting time of 35 seconds often used for sport horses. It can be used with pad or mesh. For the best result it is advisable to use a mesh.

Comes in 250cc cartridges fitting in standard caulking guns. Fill up to ground level for support of foundered and laminitis cases. Usage of meshes is recommended, but not obliged. Set time approx. 35 seconds. Temperature and humidity might have effect on the performance of this product.

PRODUCT APPLICATION

Standard application

- With pads

Alternative application

- With mesh

UPC



TECHNICAL PRODUCT DATA

Product Number	200150250
Description	Medium hard & supportive pad material with adhesion
Cartridge	250 cc
Color	Cristal Clear
Set time	35 seconds

Glue-U Adhesives BV

Droogdokkenland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN

V.003.US

SHUPACK

CS A50

- Medium hard shock absorbing material
- Stays stable during shoeing period, even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time
- Protects frog against thrush



SHUPACK CS is a shock absorbing pad material with a setting time of 35 seconds often used for sport horses. It can be used with pad or mesh. For the best result it is advisable to use a mesh.

Comes in 250cc cartridges fitting in standard caulking guns. Fill up to ground level for support of foundered and laminitis cases. Usage of meshes is recommended, but not obliged. Set time approx. 35 seconds. Temperature and humidity might have effect on the performance of this product.

PRODUCT APPLICATION

Standard application

- With pads

Alternative application

- With mesh

UPC



TECHNICAL PRODUCT DATA

Product Number	200152250
Description	Medium hard & supportive pad material with adhesion and extra protection against thrush
Cartridge	250 cc
Color	Green
Set time	35 seconds

Glue-U Adhesives BV

Droogdokkenland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN
V.003.US

SHUGUARD

A60

- Hard supporting pad material
- Stays stable during shoeing interval, even in cold weather conditions
- Adhesion to the sole
- 35 seconds setting time
- Becomes hot during setting



SHUGUARD is a supporting pad material which can be used on horses with shoes or barefoot. Thanks to the support on the sole it relieves the pressure on the wall and improves heel growth.

Comes in 250cc cartridges fitting in standard caulking guns. Set time approx. 35 seconds. Temperature and humidity might have effect on the performance of this product.

PRODUCT APPLICATION

Standard application

- No pads needed, possible with mesh

UPC



TECHNICAL PRODUCT DATA

Product Number	200161250
Description	Supportive pad material with adhesion
Cartridge	250 cc
Color	Black
Set time	35 seconds

Glue-U Adhesives BV

Droogdokkenland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN

V.003.US

SHUSIL

SOFT A10

- Very soft filling material
- Stays soft even in cold weather conditions
- No adhesion to the sole 4 minutes setting time
- If heated upto body temperature, setting time becomes 2 minutes
- Keeps low temperature during setting
- Acid free



SHUSIL A10 is a silicone product to be used as hoofpad material. Can be used as filling or light shock absorbtion.

250cc cartridges fitting in standard caulking guns. Offers protection and light shock absorbtion for the hoof. Always use SHUSIL in combination with Shupads! SHUSIL does not contain acids or chemicals and are therefore very suitable in combination with wound treatment. Setting time 4 minutes, depending on weather conditions. Heating up SHUSIL before use speeds up setting time significantly. Temperature and humidity might have effect on the performance of this product.

PRODUCT APPLICATION

Standard application

- Always with pads

UPC



TECHNICAL PRODUCT DATA

Product Number	210105250
Description	Extremely soft silicone pad material
Cartridge	250 cc
Color	Blue
Set time	4 minutes (heated 2 minutes)

Glue-U Adhesives BV

Droogdokeneiland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN
V.003.US

SHUSIL

MEDIUM SOFT A20

- Soft filling / light shock absorbing material
- No adhesion
- Stays soft even in cold weather conditions
- 4 minutes setting time (2 minutes when heated)
- Keeps low temperature during setting
- Acid free



SHUSIL Medium Soft A20 is a transparent silicone product to be used as hoofpad material. It can be used as filling or shock absorption.

250cc cartridges fitting in standard caulking guns. Offers protection, shock absorption and support for the hoof. Always use SHUSIL in combination with Shupads! SHUSIL does not contain acids or chemicals and are therefore very suitable in combination with wound treatment. Setting time 4 minutes, depending on weather conditions. Heating up SHUSIL before use, speeds up setting time significantly. Temperature and humidity might have effect on the performance of this product.

PRODUCT APPLICATION

Standard application

- Always with pads

UPC



TECHNICAL PRODUCT DATA

Product Number	210103250
Description	Medium soft silicone pad material
Cartridge	250 cc
Color	Transparent
Set time	4 minutes (heated 2 minutes)

Glue-U Adhesives BV

Droogdokkenland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN

V.003.US

SHUSIL

MEDIUM A30

- Medium shock absorption material
- No adhesion
- Stays soft even in cold weather conditions
- 4 minutes setting time (2 minutes when heated)
- Keeps low temperature during setting
- Acid free



SHUSIL Medium A30 is a silicone hoofpad material, mostly used for shock absorption. It offers more comfort to sport horses and also reduces stress on the feet for foundered horses.

250cc cartridges fitting in standard caulking guns. Offers protection, shock absorption and support for the hoof. Always use SHUSIL in combination with Shupads! SHUSIL does not contain acids or chemicals and are therefore very suitable in combination with wound treatment. Setting time 4 minutes, depending on weather conditions. Heating up SHUSIL before use, speeds up setting time significantly. Temperature and humidity might have effect on the performance of this product.

PRODUCT APPLICATION

Standard application

- Always with pads

UPC



TECHNICAL PRODUCT DATA

Product Number	210109250
Description	Medium silicone shock absorbing pad material
Cartridge	250 cc
Color	Purple
Set time	4 minutes (heated 2 minutes)

Glue-U Adhesives BV

Droogdokkenland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN

V.003.US

SHUDIM AIR

ULTRA LIGHT A20

- Ultra light pad material
- Non toxic
- Setting time 4-5 min (heated 2-3 min)



SHUDIM Air is an ultra light impression material to be mixed by hand. It's mostly used for filling, light shock absorption or just to give more comfort to the horse. Since the product is very soft it is only possible to fill complete hoofs.

Lightweight impression material for shock absorption and protection. Easily mixable and shapeable by hand. The ultra light SHUDIM Air is available in 2100 cc in durometer A20 and weighs 35% less than any standard impression material in the industry. SHUDIM is a non irritating product. Easy to cut which makes it applicable in many situations. SHUDIM does not contain acids or chemicals and is very suitable in combination with wound treatment. Temperature and humidity might have effect on the performance of this product. Heating up SHUDIM before use speeds up setting time significantly.

PRODUCT APPLICATION

Standard application

- Always with pads

UPC



TECHNICAL PRODUCT DATA

Product Number	250114140
Description	Ultra light putty pad material mixed by hand
Jar / Bucket	2100 cc + 3x23 cc / 3.08 lb + 3x23 cc
Color	Orange
Set time	4-5 minutes (heated 2-3 minutes)

Glue-U Adhesives BV

Droogdokeneiland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN

V.003.US

SHUDIM

SUPER SOFT A15

- Very soft impression material
- Non toxic
- Skin friendly (not irritating or causing allergic reactions)
- Setting time 4-5 min (heated 2-3 min)
- Easily mixable and shapeable by hand
- Can be combined with any medical application to the hoof



SHUDIM Super Soft A15 is an extra soft impression material to be mixed by hand. It's mostly used for filling, light shock absorption or just to give more comfort to the horse. Since the product is very soft it is only possible to fill complete hoofs. In case of laminitis or other cases where there is an extremely painful area of the sole that needs to be isolated, the impression material can be trimmed away once it is cured after applying the shoe and pad.

Lightweight impression material for shock absorption and protection. Easily mixable and shapeable by hand. SHUDIM is a non irritating product. Easy to cut which makes it applicable in many situations. SHUDIM does not contain acids or chemicals and is very suitable in combination with wound treatment. Temperature and humidity might have effect on the performance of this product. Heating up SHUDIM before use speeds up setting time significantly.

PRODUCT APPLICATION

Standard application

- Always with pads

Alternative application

- Medical plate

UPC



TECHNICAL PRODUCT DATA

Product Number	250105660 (1.5 lb) 250105025 (5.5 lb)
Description	Extra soft putty pad material mixed by hand
Jar / Bucket	2x1.5 lb 2x5.5 lb
Color	Blue
Set time	4-5 minutes (heated 2-3 minutes)

Glue-U Adhesives BV

Droogdokkenland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN

V.003.US

SHUDIM

SOFT A25

- Soft impression material
- Non toxic Skin friendly (not irritating or causing allergic reactions)
- Setting time 4-5 min (heated 2-3 min)
- Easily mixable and shapeable by hand
- Can be combined with any medical application to the hoof



SHUDIM Soft A25 is a soft impression material and it is the most used version of this product range. It gives a perfect shock absorption to the complete sole or locally as frog support in combination with for example a heartbar shoe.

Lightweight impression material for shock absorption and protection. Easily mixable and shapeable by hand. SHUDIM is a non irritating product. Easy to cut which makes it applicable in many situations. SHUDIM does not contain acids or chemicals and is very suitable in combination with wound treatment. Temperature and humidity might have effect on the performance of this product. Heating up SHUDIM before use speeds up setting time significantly.

PRODUCT APPLICATION

Standard application

- Always with pads

Alternative application

- Locally on bar shoes (Hearthbar, Straightbar)

UPC



TECHNICAL PRODUCT DATA

Product Number	250106660 (1.5 lb) 250106025 (5.5 lb)
Description	Soft putty pad material mixed by hand
Jar / Bucket	2x1.5 lb 2x5.5 lb
Color	Pink
Set time	4-5 minutes (heated 2-3 minutes)

Glue-U Adhesives BV

Droogdokeneiland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN

V.003.US

SHUDIM

MEDIUM A35

- Medium impression material
- Non toxic Skin friendly (not irritating or causing allergic reactions)
- Setting time 4-5 min (heated 2-3 min)
- Easily mixable and shapeable by hand
- Can be combined with any medical application to the hoof



SHUDIM Medium A35 is your best partner to support the hoof completely or just locally at the frog. Thanks to the rigid end product it's very easy to use with all kinds of barshoes or pads.

Lightweight impression material for shock absorption and protection. Easily mixable and shapeable by hand. SHUDIM is a non irritating product. Easy to cut which makes it applicable in many situations. SHUDIM does not contain acids or chemicals and is very suitable in combination with wound treatment. Temperature and humidity might have effect on the performance of this product. Heating up SHUDIM before use speeds up setting time significantly.

PRODUCT APPLICATION

Standard application

- Always with pads

Alternative application

- Locally on bar shoes (Hearthbar, Straightbar)

UPC



TECHNICAL PRODUCT DATA

Product Number	250107660 (1.5 lb) 250107025 (5.5 lb)
Description	Medium putty pad material mixed by hand
Jar / Bucket	2x1.5 lb 2x5.5 lb
Color	Green
Set time	4-5 minutes (heated 2-3 minutes)

Glue-U Adhesives BV

Droogdokeneiland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN
V.003.US

SHUFIX

WHITE

- Super fast setting time (30 - 60 seconds)
- Easy to rasp
- Hard but flexible material
- High temperature while setting (keep +/- 0.4") from coronary band)



SHUFIX White is an ultra fast urethane glue which is very suitable for making extensions on foals' hoofs and custom plastic shoes. Also suitable for retouching or rebuilding heels, making medial/lateral corrections and for smaller hoof repairs. In severe winter circumstances SHUFIX can also be used to glue most kind of glue-on shoes. Since it's a urethane product, the hoof should be dry and clean before applying SHUFIX.

250cc cartridges fitting in standard caulking guns. Curing time between 30 - 60 seconds, workable after about 5 minutes. Temperature and humidity can influence the performance of this product.

PRODUCT APPLICATION

Standard application

- Hoof extensions

Alternative application

- Glue-on shoes
- Crack repairs
- Temporary protection
- Heel lifts and medial/lateral corrections

UPC



TECHNICAL PRODUCT DATA

Product Number	150101250
Description	Instant crack repair super fast setting / foal extensions
Cartridge	250 cc
Color	White
Set time	30-60 seconds

Glue-U Adhesives BV

Droogdokkenland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN

V.003.US

SHUBOND BLACK

- 150cc cartridges usable in standard standard caulking guns
- Very high adhesion
- Easy to rasp
- Hard but flexible material
- Setting time can be speed up when heated to 86 Fahrenheit



SHUBOND Black acrylic glue is the most versatile glue used in the horse industry. It can be used to glue-on every kind of shoe or do repairs from the smallest crack to the most extreme hoof wall problems.

Available in 150cc and 420cc black. Fits in a standard caulking gun using an adapter or just use the Glue-U dispenser for 150 cc cartridges. 420cc only usable with special gun. Bonds steel, aluminium and any kind of glue-on shoe. Suitable for repairing large cracks and creating extensions.

Initial set time: 1.5-2 minutes (open time). Weight-bearing in 4-6 minutes (even in cold conditions). Temperature and humidity might have effect on the performance of this product. Heating up SHUBOND before use, speeds up setting time significantly.

PRODUCT APPLICATION

Standard application

- Glue-on shoes

Alternative application

- Crack repairs
- Hoof repairs

UPC



TECHNICAL PRODUCT DATA

Product Number	100102150 (150 cc) / 100102420 (420 cc)
Description	Glue-on shoe / Hoof repair
Cartridge	150 cc 420 cc
Color	Black
Set time	Initial set time: 1.5-2 minutes / Weight bearing: 4-6 minutes / Heated: 3 - 4 minutes

Glue-U Adhesives BV

Droogdokeneiland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN
V.003.US

SHUBOND

BEIGE FAST SET

- Fast setting time / Weight bearing 3-4 minutes
- Easy to rasp
- Hard but flexible material
- Setting time can be speed up when heated to 86 Fahrenheit



SHUBOND Beige acrylic glue is the most versatile glue used in the horse industry. It can be used to glue-on every kind of shoe or do repairs from the smallest crack to the most extreme hoof wall problems.

Available in 150cc and 420cc beige. Fits in a standard caulking gun using an adapter or just use the Glue-U dispenser for 150 cc cartridges. 420cc only usable with special gun. Bonds steel, aluminium and any kind of glue-on shoe. Suitable for repairing large cracks and creating extensions.

Initial set time: 1.5-2 minutes (open time). Weight-bearing in 3-4 minutes (even in cold conditions). Temperature and humidity might have effect on the performance of this product. Heating up SHUBOND before use speeds up setting time significantly.

PRODUCT APPLICATION

Standard application

- Glue-on shoes

Alternative application

- Crack repairs
- Hoof repairs

UPC



TECHNICAL PRODUCT DATA

Product Number	100114150 (150 cc) / 100114420 (420 cc)
Description	Glue-on shoe / Hoof repair
Cartridge	150 cc 420 cc
Color	Beige
Set time	Initial set time: 1.5-2 minutes / Weight bearing: 3-4 minutes / Heated: 2-3 minutes

Glue-U Adhesives BV

Droogdokeneiland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN
V.003.US

PRE FIX ACTIVATOR AND CA GLUE

- 5 seconds fast setting superglue
- Bonds once 2 components are in contact



PRE FIX is a system to bond a GLUSHU to the foot just to keep it in place. It is a 2 component superglue which will bond the GLUSHU to the hoof in seconds. This allows you to do the actual glueing of the shoe with the foot on the ground. PRE FIX is very useful if you have a very nervous horse or a horse that is unable to stand on three legs (laminitis or injury). PRE FIX is no structural glue. It is temporarily only to keep the GLUSHU in place for the time you need until the actual glue is cured. One component is the activator spray, the other component is a transparent gel in a squeeze bottle.

PRE FIX is used in 2 steps. Once you fitted and shaped the GLUSHU and you are ready to glue, apply the PRE FIX CA GLUE on the GluShu and spray the activator on the hoof. After pre fixing the shoe you can further finish the glueing on the ground with SHUBOND for permanent fixation.

PRODUCT APPLICATION

Standard application

- Only for pre-application of GLUSHU shoes

UPC



TECHNICAL PRODUCT DATA

Activator

Product Number	490100200
Description	Pre glue GLUSHU in combination with PRE FIX CA GLUE
Cartridge	200 cc
Color	Transparent
Set time	5 sec

CA Glue

Product Number	490101150
Description	Pre glue GLUSHU in combination with PRE FIX ACTIVATOR

Glue-U Adhesives BV

Droogdokkenland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN

V.003.US

Cartridge	150 cc
Color	Transparent
Set time	5 sec

Glue-U Adhesives BV

Droogdokkeneiland 8
5026 SR Tilburg
The Netherlands

Tel. +31 (0)13 545 31 18
E-mail: sales@glue-u.com
Website: www.glue-u.com

Glue-U USA Inc

215 Church Street
Georgetown
SC 29440 USA

Tel. +1 (313) 509 - 7766
E-mail: sales-us@glue-u.com
Website: www.glue-u.com

EN
V.003.US